

MC-D01



On basis of the international popular concept of obstetric integration, this Mcare LDR bed is designed to provide a comprehensive solution for the entire period of the delivery.

Prominent features make the Mcare bed totally different from the delivery table and other similar tables, it is wider enough to provide comfort, and the very flexible foot stand leg support can be concealed underneath which converts from a normal bed to a delivery table immediately.

Main features

- It is easy to operate, and very flexible to move around. The center dynamic system make the table more stable.
- The continued electrical supply system can be used in any emergency.
- The hidden obstruction guard can make the bed move through the narrow corridor flexibly and can move patient to the ambulance without any delay.
- It has the convenient hand-switch which can be controlled by nurses and patient.

- The inductive light can lighten the shadow of the bed automatically.
- The leg support can be hidden without movement, and can be adjusted flexibly to lighten the stress of the nurses.
- It has the CPR heart and lungs-recovering function.
- The urethane bed plate is soft and convenient to wash which has the function of deodorization and antiseptis.



Main function

Semi-Fowlers position

When in the birthing process, the bed can quickly be transformed into semi-Fowlers position. Remove the foot section, place the dirt basin, and set up the leg supports. No need to move the mother



The Throne Position

The unique design of this delivery bed is that the foot section can be lowered to suitable position for the mother to place her feet on the foot plate to achieve throne position

15° pelvic tilt tilted seat designed to cradle mom to increase her feeling of security




Trendelenburg position

The Trendelenburg position can be adjusted manually, and in Trendelenburg, the patient lies supine on the bed with the head lowered to the feet at an angle. This position is mainly used for lower abdominal examination, as well as to prevent hypotension and shock, etc.




Side rail control

The side rail has clearly-icon buttons for adjustment functions, back leg and height adjustment on the outer side of the side rail, and also has lock and battery reminder.

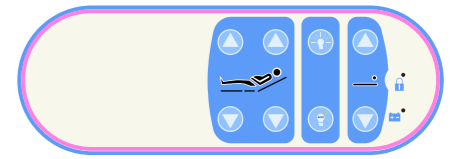

The inner side rail has the function adjustment of the back and the legs, and has the adjustment of music and sound intensity, which is very user-friendly

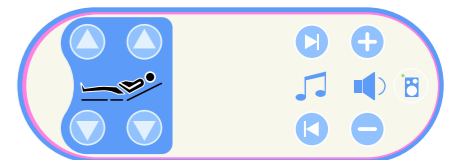

Technical Specification

ITEM	PARAMETERS
Bed dimension including bed end and bumper wheel:	2350*940mm
Bed dimension:	1950*880mm
Bed height:	540-890mm
Back section turning:	≥65° electric
Seat section turning:	≥12° electric
Trendelenburg:	≥8° manual
Leg section lifting distance:	110mm electric
Footrest swing out:	≥50° manual
Footrest folding:	≥75° manual
Guard rail up/down movement:	360mm manual
Castor	Dia 125mm

No.3 Jinxiu Road, Jinfeng Industrial Zone, Jinfeng Town, Zhangjiagang, Jiangsu Province, China

+86 13962231292
info@medik-medical.com

